INSTRUCTIONS FOR USE

CAUTION

EndoClot™ Catheter should only be used by physicians or other licensed practitioners.

DESCRIPTION

This device is designed to be used as a replacement of original catheter attached to the applicator of EndoClot™ Polysaccharide Hemostatic System (EndoClot™ PHS) when it is kinked or occluded.

INDICATIONS FOR USE

EndoClot™ Catheter should be used in conjunction with EndoClot™ PHS to spray the hemostatic powder in gastrointestinal tract.

INSTRUCTIONS FOR USE

1. Screw the catheter firmly to the applicator.
2. Insert EndoClot™ Catheter into the endoscope and position towards the lesion.

WARNING

EndoClot™ Catheter should be used with applicator of EndoClot™ PHS.

EndoClot™ Catheter is supplied sterile for SINGLE USE ONLY. Do not resterilize and/or reuse this device.

Use this device immediately after the package has been opened.

Do not use if the package is opened or damaged when received.

Store in a dry location under room temperature, do not expose to temperature range outside -40°C to 60°C during transportation.


This product should be disposed of in compliance with pertinent government regulations regarding medical devices.

ADVERSE REACTIONS

None reported to date.

HOW SUPPLIED

EndoClot™ Catheter		
EPAA180	1800 mm	5pcs/box
EPAA230	2300 mm	5pcs/box

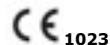©2013 All rights reserved
 Manufactured by: EndoClot Plus, Inc.
 5201 Great America Parkway, Suite 526
 Santa Clara, CA 95054, USA
 Tel: (408) 980-9125 Fax: (408) 980-9237
 E-mail: info@endoclot.com
 Website: www.endoclot.com

Authorized European Union Representative
 EC Representative: ClotPlus Limited
 Regus House, Block 4, Harcourt Road
 Dublin 2, Ireland
 Tel: +353 (0)1 477 3466 Fax: +353 (0)1 402 9590
 E-mail: info@clotplus.com

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